## ENZYMAX LIQUID (IMS-1224, IMS-1226)

**Application:** Enzymatic Cleaner liquid

**Net Contents:** IMS-1224: Quart Bottle, 950 ml with pump  
IMS-1226: 3,8 liters bottle

**Instructions for use:** Shake well before using. Wear gloves and eye protection. Use with water 32°-60°C, demineralized water is ideal. **Pre-soak:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. Soak for 3-5 minutes. Rinse, dry thoroughly and proceed with sterilization. **Ultrasonic Cleaning:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. After cleaning, rinse, dry thoroughly and proceed with sterilization. **Evacuators:** Add 60ml per 3.8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.  
- Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day

### DANGER

**H319** Causes serious eye irritation. **H334** May cause allergy or asthma symptoms or breathing difficulties if inhaled. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P337+P313** If eye irritation persists: Get medical advice/attention. **P342+P311** If experiencing respiratory symptoms: Call a POISON CENTER/doctor. Dispose of contents/container in accordance with local/regional/national/international regulations. **Safety Information:** Contains Subtilisin, Amylase, alpha- May produce allergic reaction.

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