ENZYMAX LIQUID CONCENTRATE (IMS-1222)

Application: Enzymatic Cleaner liquid

Net Contents: IMS-1222: Box of 40 Liquid Packets of 10 ml (concentrate)

Instructions for use: Shake well before using. Wear gloves and eye protection. Use with water 32°–60°C, demineralized water is ideal. **Pre-soak:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. Soak for 3–5 minutes. Rinse, dry thoroughly and proceed with sterilization. **Ultrasonic Cleaning:** Add 30ml of concentrate per 3.8L of water. Bottles with pump insert: pump one time per liter of water. Single packet: 1pck (10ml) per 3,5 liter of water. After cleaning, rinse, dry thoroughly and proceed with sterilization. **Evacuators:** Add 60ml per 3.8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

- Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day

DANGER

H318 Causes serious eye damage. **H334** May cause allergy or asthma symptoms or breathing difficulties if inhaled. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue

rinsing. **P310** Immediately call a POISON CENTER/doctor. **P342+P311** If experiencing respiratory symptoms: Call a POISON CENTER/doctor. Dispose of contents/container in accordance with local/regional/national/international regulations. **Safety Information:** Contains Subtilisin, Amylase, alpha- May produce allergic reaction.

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