(GB) ENZYMAX EARTH POWDER PAX (IMS-1332,IMS-1333 IMS-1332SP)**Application:** Enzymatic Cleaner on Powder base**Net Contents:** IMS-1332: 32 Powder packets 14 g (water-soluble)

IMS-1333: 96 Powder packets 14 g (water-soluble) - IMS-1332SP: 3 Powder packets 14 g (water-soluble)

Instructions for use: Wear gloves and eye protection. Use with water 32°-60°C, demineralized water is ideal. **Presoak:** Add 1 packet per 3.8L of water. Soak for 3-5 minutes. **Linen Presoak:** Add 1 to 2 packets per 3.8L. **Ultrasonic Cleaning:** Add 1 packet per 3.8L of water. Rinse thoroughly, dry and proceed with sterilization. **Evacuators:** Add 2 packets per 3.8L of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly. Do not use disinfectants together with Enzymax. - Do not use Enzymax in the Thermodesinfector unit. Change solution at least once a day.**DANGER****H315** Causes skin irritation. **H318** Causes serious eye damage. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.**Safety Information:** **EUH208** Contains Subtilisin. May produce an allergic reaction. Dispose of contents/container in accordance with local/regional/national/international regulations.**SDS** available on request or at: www.hu-friedy.eu**24H Emergency number:** INTERNATIONAL +49 - (0) 6132 - 84463, GBK GmbH (24h - 7d/w - 365d/a) / In England and Wales: NHS Direct - 0845 4647, In Scotland: NHS 24 - 08454 24 24 24**(FR) ENZYMAX EARTH PAX (IMS-1332,IMS-1333 IMS-1332SP)****Application:** désinfectant enzymatique en poudre**Volume net:** IMS-1332: 32 paquets de 14 g de poudre (soluble dans l'eau) -

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